

1

PRIMARY WOUND DRESSING


1.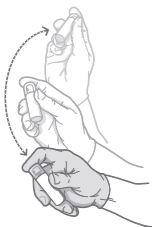

2.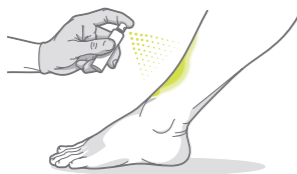

3.

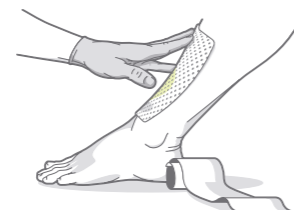EN

Product description

Primary wound dressing (spray) for acute and chronic wounds.

- 1 contains St. John's wort oil (Oleum Hyperici), Neem oil (Oleum Azadirachtae), propane, butane.
- 1 is 100% natural.
- 1 maintains a moist wound environment and enables the body to activate physiological wound healing.
- 1 protects the surrounding skin against maceration and supports healthy skin function.
- 1 prevents the secondary dressing from adhering to the wound, thereby reducing skin trauma and pain during dressing change.
- 1 has an antimicrobial effect.
- 1 is not irritating.

Instructions for use

Adhere to local hygiene measures and wound cleansing techniques according to standard medical practice prior to the application of 1.

- Shake well before use.
- Spray 1 directly on the wound bed and the surrounding skin area from a distance of 5–10 cm. Ensure that the wound and surrounding skin are covered with a clearly visible oil layer.
- Cover the wound with a suitable secondary dressing (e.g. with a non-woven gauze or absorber adjusted to the wound exudate).

Apply more oil if the secondary dressing adheres to the wound. It is recommended to repeat the treatment every 24 to 48 hours.

Contraindications

- Do not use on individuals who have had an allergic reaction to any of the ingredients of 1.

Precautions

- For external use only.
- Adjust number of puffs to wound size. Do not apply more than 20 puffs per day.
- Since no data on the safe use in children below 12 years of age and in pregnant and nursing women is available, the use of 1 in these patient groups is not recommended.
- Avoid contact with eyes and mucous membranes.
- Avoid direct contact between the wound and the spray nozzle.
- Do not inhale the spray mist.
- After repeated use for 30 days interrupt the treatment for 2 days.
- In case of adverse effects such as skin irritation or allergic reactions, remove the dressing and rinse off the oil layer with saline solution or water.
- A potential interaction between 1 and other topical treatments has not been assessed.
- Pressurized container: protect from sunlight and do not expose to temperatures exceeding 50°C.
- Do not pierce or burn, even after use.
- Do not spray on a naked flame or any incandescent material.
- Keep away from sources of ignition – No smoking.
- Keep out of the reach of children.

Storage and disposal

Store 1 at room temperature (15–25°C). Dispose according to local environmental regulations. Do not use 1 after the expiry date marked with  on the aerosol can and package.

www.1wound.info

Phytoceuticals Ltd, Zollikerstr. 44
8008 Zurich, Switzerland

The package insert was last revised on July 30, 2013.

The symbol 3 declares conformity with directive 75/324/EEC relating to aerosol dispensers. The symbol  indicates the total capacity of the aerosol container of 25ml.

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